



**THE
ROTUNDA
HOSPITAL**
DUBLIN

Umbifunnel – An umbilical cord blood collection device that is versatile, safe, efficient and user friendly.

Umbifunnel Instructions for Use:

1. The device nozzle is inserted into the required variety of laboratory tube (dependent on indication for cord blood sample).
2. The cut end of the umbilical cord is inserted into the receiving chamber of the device.
3. One hand is used to hold the UmbiFunnel device/connected tube while stabilising the inserted cord with the thumb of that same hand. The user's free hand is used to 'milk' the cord blood sample into the UmbiFunnel/connected tube.
4. When a sufficient volume of blood has been collected, the tube is removed from the UmbiFunnel device and capped.
5. The user's gloves can then be changed and the tube can be labelled and sent to the medical laboratory for analysis.

For further Information and to view the Umbifunnel instructional video please visit www.keyplastics.ie